Expedition Plus

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Stand: 02/2015

Introduction

The Expedition Plus Transport Chair is made with a lightweight aluminum frame. It is light and can be easily transported in a car or on trains.

The transport chair is ideal for occasional use indoors and outdoors with appropriate flooring or pavement (e.g., shopping center, airport).

The maximum user weight is around 130 kg (ca. 286 pounds). An accompanying person is necessary for pushing the chair when in use.

The Expedition Plus transport chair comes with a built-in seat belt.

Scope of delivery contents

1. Transport Wheelchair Expedition Plus
2. Operating instructions

Assembly instructions

1. Unfold the transport chair by pushing the 2 sides of the sitting surface down. Ensure that the transport chair is completely unfolded.
2. Pull the pushing handles up until there is an audible snap. It is not securely locked until you hear this.
3. Mount the foot rests as illustrated (picture below).

Folding and transport

Note: Fold the chair by following the unfolding directions in reverse order.

1. Press the release lever for the backrest and fold it downwards as pictured.

2. Take the foot rests off.
3. Lift the transport chair by the seat and the chair will automatically fold up.
4. The Expedition Plus is **not suitable** for use as a seat in a motor vehicle. Users should always use the vehicle's seat!

Footrest adjustment

Press the pins from both sides to adjust the foot rests.

Parking brakes

The Expedition Plus comes with two parking brakes which may only be used when the chair is standing still. To operate the parking brakes, press the brake handle down as shown in the photo. To release the parking brakes, pull the brake handles back up.

Operation and safety

- The Expedition Plus Transport Chair is suitable for persons up to a maximum of 130 kg (286 pounds) in weight and must be pushed by an accompanying person.
- The tip assist can be used to navigate curbs or door thresholds. To do this, step on the tip assist with a foot and pull on the pushing handles at the same time to lift the front of the transport chair up.
- The Expedition Plus is not suitable for going over curbs and obstacles greater than 5 cm without the help of an additional person.
- Push slowly when going up and down slopes. Do not use the Expedition Plus on down slopes of more than 10°.
- Do not use on escalators.
- Do not lean out of the transport chair! Danger of tipping over!
- Do not reach for objects out of the reach of your arm's length.
- Avoid hazardous places such as narrow door openings, steps, household appliances, children's toys on the ground, etc.
- Do not set anyone in the transport chair from the side. Always place the person in the transport chair from the front.
- The parking brakes must not be used to reduce speed. Use them only as brakes when the chair is standing still.

Care and maintenance

- The aluminum frame of the Expedition Plus Transport Chair and the nylon cushions can be cleaned with a moist cloth and mild household cleaners and disinfectants.
- After contact with moisture, always rub the transport chair dry (spot formation).
- The transport chair should be inspected monthly for safety. All screws that may be loose must be re-tightened, and damaged or worn parts such as covers must be replaced immediately.
- The transport chair is delivered with wheels of solid plastic material. No pumping is necessary.
- The transport chair is suitable for reuse. Ask the manufacturer about the requirements for reuse.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

This service life may even be exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology. The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Reprocessing

The transport chair is suitable for reuse.

Ensure that the new user is given all of the technical documents required to ensure safe handling.

The transport chair must be cleaned and disinfected prior to reuse and must be inspected for damage and approved by the retailer.

Warranty

The warranty is 24 months from the date of purchase.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

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Manufacturer: Drive Medical GmbH & Co. KG • Leutkircher Straße 44 • D-88316 Isny/Allgäu • Germany •
Distributor: DeVilbiss Healthcare GmbH • Kamenzer Straße 3 • D-68309 Mannheim • Germany •